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Product: Needle for microdissection Traumec

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READ USER MANUAL BEFORE USING THE PRODUCT
PRODUCT USE ONLY - FORBIDDEN REPROCESSING
STERILE PRODUCT - sterilized ETHYLENE OXIDE
Do not use if package is damaged or tampered with or if the
product has expired.

Store and transport the product in clean, dry place away from heat and protected from direct light, in Temperature: +10 ° to +40 ° C -
Relative Humidity: 85% maximum

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1.1 - Name and model

Technical name: Disposable Electrode

Trade Name: Needle microdissection Traumec

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1.2 - Description of product:

The needles for microdissection Traumec are products produced for to be used with Electrosurgical Equipment and their pens. They are presented in various forms which allows the surgeon better performance in surgery. All models are provided with tip of tungsten, which allows a better quality product.

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1.3 - The physical principle and foundations of product technology, applied for its operation and its action:

The electrode connected to the electrosurgery unit, driving energy between the electrosurgery apparatus and the patient, thereby cutting tissue, cauterizing vessels while the blood to prevent excessive bleeding.

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1.4 - Technical Specifications and Features:

The electrode consists of a tip in Tungsten, body in AISI 304 stainless steel coated with high temperature resistant polymer.

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2 - INDICATION, PURPOSE OR INTENDED USE OF THE PRODUCT:

Conduction of energy between the electrosurgery device and the patient, ie, disposable electrode is responsible for the actions of cutting, cautery dissection and soft tissue.

The compatibility of microdissection needles Traumec with electrosurgical devices and their pens must be secured before use.

The Traumec microdissection needles were tested with the following equipment Electrosurgical and his pen: Electrosurgical Source: Make: Model DELTRONIX: B MP 1600 Monopolar Pen: Make: Model DELTRONIX: CM-05

Note: Other equipment and other electrosurgical pens can be compatible, however, before use, check the Instructions for Use of these products (equipment), and its specifications and recommendations, or contact our technical department for further guidance.

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3 - WARNINGS AND / OR PRECAUTIONS TO BE TAKEN

Before using, make sure that the electrode is connected firmly to the pen. Do not use if package is damaged or tampered with or if the product is expired. Single-use product, do not reuse. Do not use if there are signs of damage. Handle only by the tubular electrode. Do not bend, nor straighten the electrode. Loading the electrode so as not to be clearance between the same and a handpiece. The metal tube can not come into contact with the tissue and can cause injury. Before connecting or removing the electrode, make sure that the pen is not connected to the electrosurgical unit / electric scalpel or electrosurgical unit /Electronic scalpel is OFF / or on standby. Always use generators marketed legally compatible with monopolar electrosurgical devices and legislation that meet the ANVISA. Do not connect wet accessories or internal moisture generator. There may be danger of electric shock.

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4 - AGAINST INDICATION:

This product can not be used in procedures in the central nervous system or central circulatory system.

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5 - CONDITION OF STORAGE, TRANSPORTATION AND HANDLING:

Transport and Store product in clean, dry place away from heat and protected from direct light and in their original packaging, in Temperature: +10 ° to +40 ° C - Relative Humidity: 30 to 85%. The electrode should be handled carefully and individually, avoiding bumping or

dropping. Any product that has been dropped or improperly handled, or suspected to have suffered damage, should be separated and segregated.

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6 - FORMS OF PRESENTATION OF MEDICAL EQUIPMENT

The electrodes are individually packaged in double sterinbag, properly labeled as DRC 185, 2001, and commercialized individually in final packaging, made of paper triplex.

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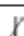
7 – INSTRUCTIONS OF USE

Opening the package by pulling the tab opening at the end of the pack and removes it from the package. Its handling should follow aseptic techniques already used in surgical centers. Connect the monopolar electrode in the pen slot compatible with the product. The electrode must be securely fastened monopolar pen, and this connected to the electrosurgical equipment. Initially you must select the power of the device between 3 and 5 Watts gradually increasing for improved performance and efficiency of the procedure.

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